# Tx360® Nasal Applicator

## INSTRUCTIONS FOR USE

The Tx360 Nasal Applicator is for Single Use Only. Not To Be Reused. Discard appropriately after single use. NOTICE: Please read the complete instructions for use prior to using the Tx360 Nasal Applicator. Failure to do so may result in patient injury.

Rx Only

EN

## INTRODUCTION

### Tx360 INTENDED USE


The Tx360 Nasal Applicator is a sterile device intended for use in adults to deliver small amounts of fluid (<1cc, luer lock) selected by a healthcare professional to the nasal pathway, including the area of the inferior turbinate, superior turbinate, sphenopalatine foramen, etc. It is intended for use on adult patients with intact nasal mucosa.

### Tx360 NASAL APPLICATOR PRODUCT DESCRIPTION

Tian Medical LLC provides a simple and convenient nasal applicator for delivery of fluid (drugs or solutions) to the nasal cavity. The Tx360 device is compatible with aqueous and light oil solutions. The healthcare professional controls the type and amount of solution, the location, and the rate of delivery. The healthcare professional controls the rate of delivery by the pressure exerted on the plunger of the 1cc syringe. The Tx360 attaches to a provided luer lock 1cc disposable syringe allowing the healthcare professional to deliver small and measured amounts of fluid to areas along the nasal pathway. A hub with a small length of flexible, thin tubing is connected to the top of the delivery system. It has a soft rounded tip which has a single small opening to deliver the solution in a gentle stream to the desired region of the nasal pathway. All components are latex-free. The device has been designed with a slightly flared stem to assure the device cannot be placed too high in the nasal pathway. Its small flexible tip is designed to help reduce irritation to the nasal lining. Figure 1 below illustrates the device and its components.

Figure 1

### Tx360 Nasal Applicator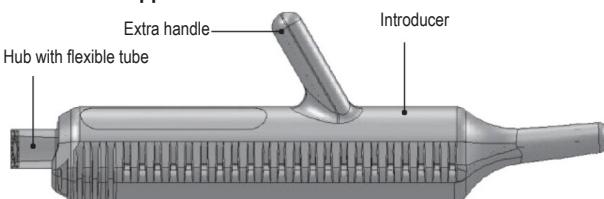

### Tx360 NASAL APPLICATOR PROVIDES

- An easy-to-use, disposable nasal applicator for Single Use Only. Not To Be Reused.
- Latex-free component.
- An extra handle for user convenience and additional support.
- Suitability for use in both right and left pathway.
- A selected volume delivered from the provided disposable 1cc syringe provides an efficient fluid stream.
- Handheld portability.
- A guide that locks to the syringe and provides a convenient method to access the desired nasal area.

### Tx360 USE AND STORAGE

- The Tx360 Nasal Applicator must be used only as directed. Please read all instructions before use.
- If you have any questions about the use of this product, contact Tian Medical LLC. Customer service at 630-426-9605.
- Single use only.

### DO NOT USE THE Tx360 NASAL APPLICATOR IN THE FOLLOWING CONDITIONS (CONTRAINDICATIONS):

- If the patient has a nasal septal deformity such as the following: malformed facial or nasal passages such as cleft lip and palate, choanal atresia (narrowed nasal passages), atropic rhinitis, rhinitis medicamentosa, septal perforation, nasal/midface trauma or he or she has recently had nasal/sinus surgery.
- Potential for injury may occur, such as a nasal passage obstruction and nasal trauma and irritation.
- If the patient has a bleeding disorder such as Von Willebrand's disease or hemophilia uncontrolled bleeding may occur.
- If the patient has severe respiratory distress.
- Further irritation of nasal airways may worsen respiratory distress.
- If the patient has a Neoplasm including, but not limited to: Angiofibroma, sinus tumors, Granuloma.
- Congestion has been present for more than 10 days, there is a high fever, and the nasal mucous is an abnormal color.

### DO NOT USE THE Tx360 NASAL APPLICATOR IF:

- The skin around and inside the nasal passage is dry, cracked, oozing, or bleeding.
- The patient experiences recurrent nose bleeds.
- The patient has a nasal or facial fracture.
- The Tx360 Nasal Applicator may cause nose bleeding and additional nasal trauma and irritation.


### WARNING AND PRECAUTIONS

The Tx360 Nasal Applicator should be used with care by healthcare professionals.

Complications may include:

- Nasal passage irritation. Putting anything in the nose may irritate the lining of the nose. To help avoid irritation, use the device for only a short duration. Do not use for more than one application. If the tissue is bright red, appears to be irritated, or if the patient is experiencing pain, immediately discontinue using the Tx360 Nasal Applicator.
- Nose Bleeding. Do not force the tip of the Tx360 into the nose. Insert the device slowly and carefully following the step by step instructions described on next page. If bleeding occurs, immediately stop using the device and apply pressure on the nose to stop bleeding.
- The Tx360 Nasal Applicator is a disposable device for Single Use Only. Not To Be Reused.
- Discard appropriately after use.
- Reuse of the Tx360 Nasal Applicator and provided 1cc syringe may cause serious infections and introduction of bacteria, including VRE (Vancomycin-Resistant Enterococcus) or MSA (Methicillin-Resistant Staphylococcus Aureus), and other serious bacteria.

### DEVICE FEATURES

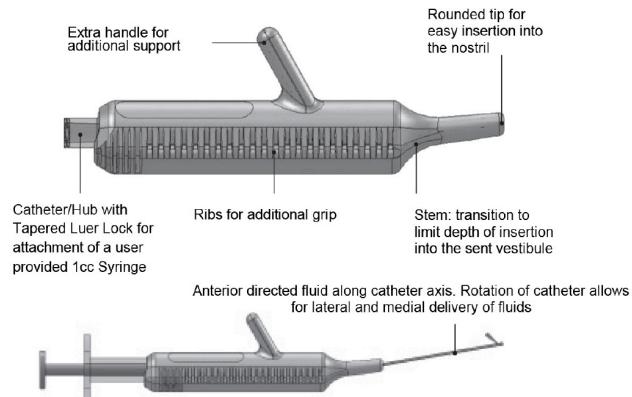Rx Only

Rx Only

Manufacturer

Do not use if package is damaged

Catalog Number

Sterilized using irradiator

Caution / Instructions for use

Batch Code

Do not reuse



Latex-free


Instructions for use


Use by date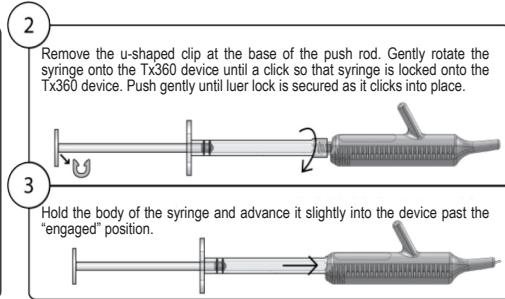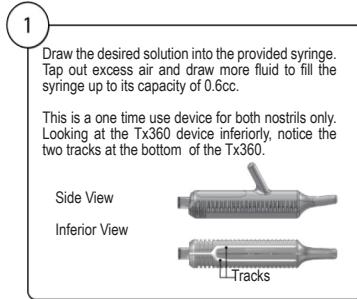

## Instructions for use of the Tx360 Nasal Applicator (Right)
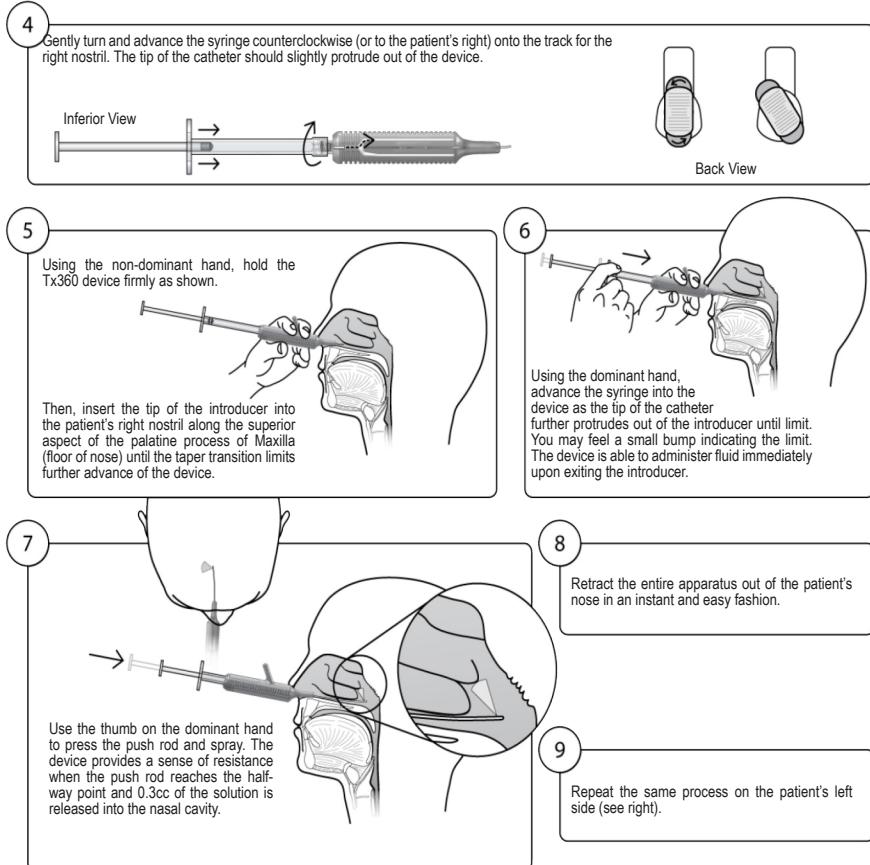
## Instructions for use of the Tx360 Nasal Applicator (Left)
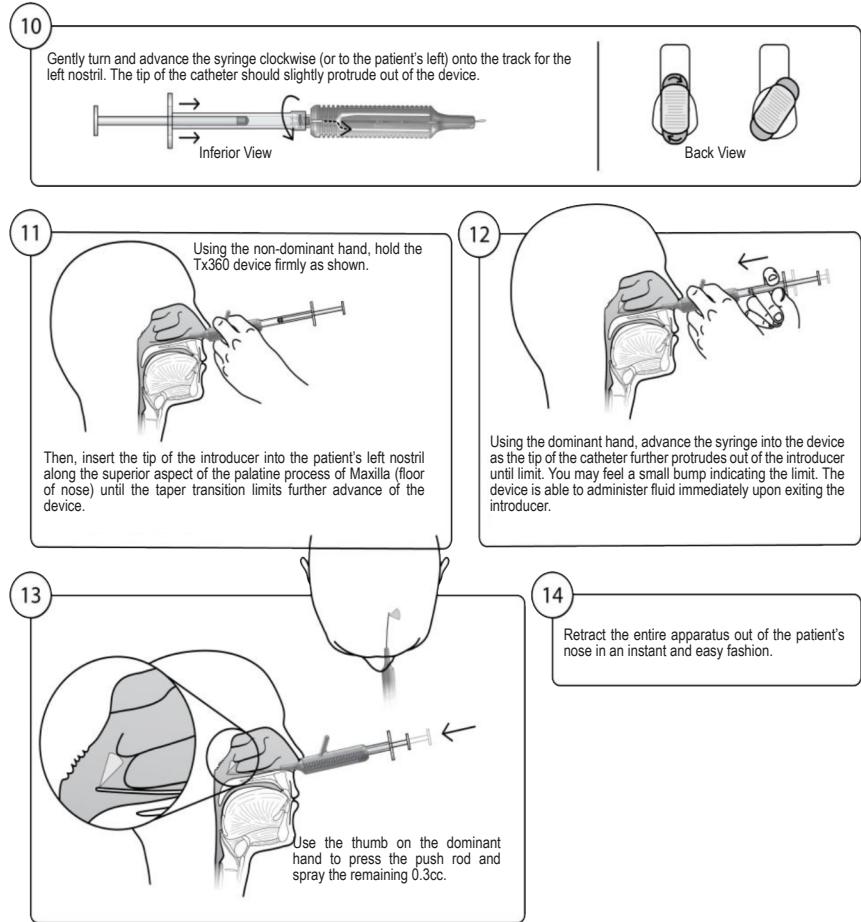
### CONTACT INFORMATION

WARNING: If you observe a defect or experience a problem with use of this device, call Tian Medical, LLC.  
 Technical Support at 855-338-0042  
 Customer Service: 630-426-9605  
 Email: sales@tianmedical.com  
 Website: www.tianmedical.com

Manufactured for Tian Medical, LLC  
 800 Liberty Dr  
 Libertyville, IL 60048 USA

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Fabricado para Tian Medical, LLC